

1 Charge Recorder

- Always keep the Recorder charging when it's not in use.
- Charge the Recorder before first use.
- Allow 12 hours for the Recorder to fully charge.
- When charging with the Recorder powered off, the Power LED will remain off but the Recorder will still be charging.
- During charging, the Recorder cannot be used on a patient.




2 Power On & Start Recording

- To power on: press the Physical **Power Button**.
- Press the **Record Button** on the home screen to start a patient recording.




3 Enter Patient Info

- Enter Patient Information, including: Patient ID, First Name, Last Name, Date of Birth, and Ordering Physician.
- Verify Patient Info entered is correct and press the **Confirm Button** to continue. Press the **Edit Button** to correct any information before continuing.

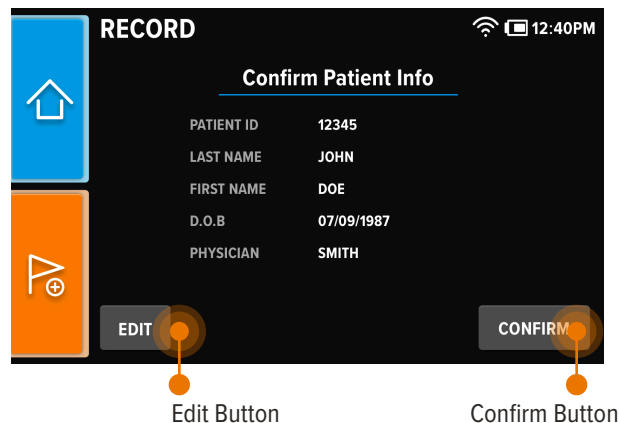

4 Setup Headband

- Connect the Headband to the Recorder and Patient and setup headband. Refer to [Ceribell EEG Headband Operator Manual](#) for additional instructions.
- Press the **Record Button** when all Electrodes are Green.

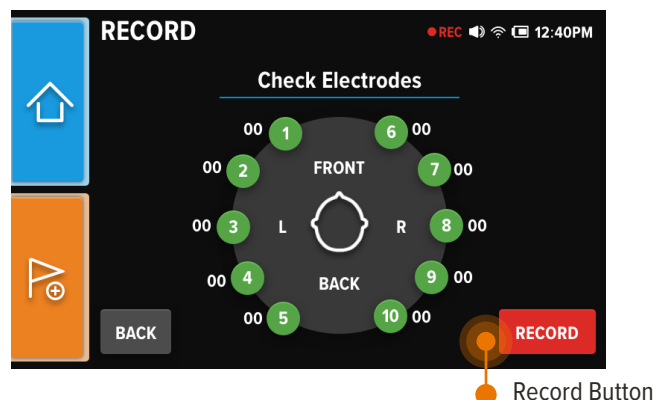

5 Brain Stethoscope

- Press the **Home Button** to return to the home screen.
- To listen to EEG waveforms with the Brain Stethoscope function, press either the **Physical Sound Button** on the front of the device or the **Sound Button** on the home screen.




6 Return to Recording Screen

- Press the **Home Button** to return to home screen.
- Press the **Record Button** to return to the recording screen.




7 Stop Recording

- Press the **Stop Recording Button** to stop the recording. A stop recording confirmation screen will display.

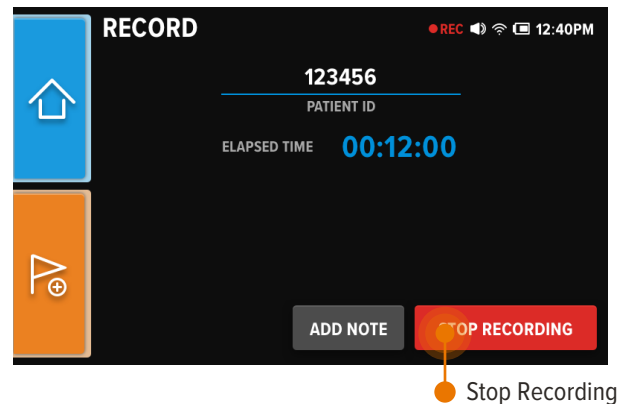

8 EEG Data Upload Through Wifi

- The Recorder may take a few moments to finish uploading the EEG data. You may now disconnect the Headband from the Recorder and patient.
- If Wifi signal is poor or the upload fails, you can stop the upload and finish the remaining percentage of the EEG data upload through USB. Please refer to [Ceribell EEG File Upload Through USB](#).

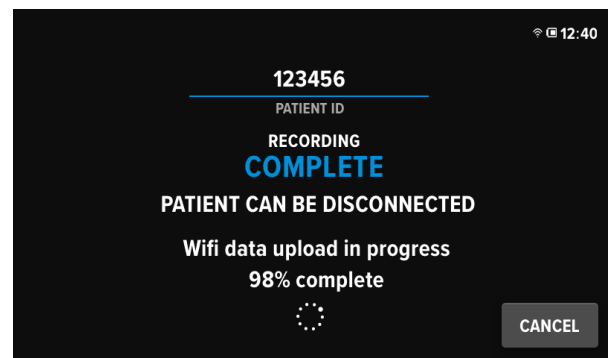

Refer to the [Ceribell EEG Recorder Operator Manual](#) for more detailed instructions and information.