

Quick Start Guide

1



CONFIRM ACTIVE CERIBELL RECORDING

- The Video Unit requires an active Ceribell EEG recording
- Locate the **Video Sync Code** on the Ceribell recorder screen

2



SET UP THE VIDEO UNIT

- Clamp the stand securely to an IV pole or stable bedside structure
- Ensure the stand is stable and will not move
- Aim camera toward patient

3



ADJUST ARM POSITION

- Adjust the Video Unit using **both hands** on the flexible arm
- Do not adjust the Video Unit by the phone or case
- Position the arm in a gentle curve toward the patient
- Avoid sharp bends (e.g., 90° or 180°)

4



POWER & CONNECT

- Plug the Video Unit into a power outlet
- Ensure the Video Unit is powered on
- Open the Ceribell Video application
- Ensure Wi-Fi is connected (**Connected** displayed)

5



ENTER VIDEO SYNC CODE

- Enter the 4-digit **Video Sync Code** (located on the Ceribell recorder "RECORD" screen)
- Video streaming starts automatically after the 4th digit is entered

6



CONFIRM VIDEO STREAMING

- Confirm recording indicator and timer are displayed (1:34:24)
- Ensure entire patient is clearly visible

7



DURING MONITORING

- Confirm Wi-Fi remains **Connected**
- Keep the Video Unit plugged in
- Ensure patient remains in frame

8



END OF SESSION

- End recording on the Ceribell Recorder
- Video will stop automatically when the session ends

Ceribell 24 Hour Customer Support: 1-800-763-0183

Troubleshooting

- Unable to sync: Confirm the Video Sync Code is entered correctly and matches the code displayed on the Ceribell EEG recorder
- No connection: Confirm the Video Unit is connected to the appropriate Wi-Fi network
- Application not active: Confirm the Ceribell Video application is open and running
- Video Unit not charging: Confirm the power cable is securely connected and plugged into an outlet
- Blurry or unclear video: Ensure the camera is unobstructed and the lens is clean

Precautions

- The Video Unit must be securely mounted to a stable surface or pole. Failure to secure the device may result in the unit falling and causing injury to the patient or damage to equipment.
- Never mount the unit directly over the patient.
- Do not place liquids on or near the Video Unit.
- Use of the Video Unit must comply with institutional policies regarding patient privacy. Ensure the camera is positioned to avoid recording unauthorized individuals or sensitive areas.
- The Video Unit may become hot to the touch during prolonged use.



Warning: Do not place or operate the Video Unit inside oxygen-enriched environments (such as neonatal incubators, oxygen tents, or pediatric hoods). Oxygen-enriched atmospheres increase the risk of fire and ignition.



Warning: Keep cables away from the patient. Cables may pose an entanglement or strangulation risk.



Scan to access the manual

DC-00458, rev A
DCO-001620 (05/2026)